**DISCO DE
RESINA
HIBRIDA**

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Inglês:

Indication:

Hybrid Resin Discs are indicated for Inlays, Onlays, full crowns for anterior and posterior teeth, implant-supported restorations and aesthetic veneers.

Instruction for Use:

The Hybrid Resin disc - Zirlux must be fixed to the support provided and cleaned in advance, in accordance with the instructions from the device manufacturer. Correct fit must be ensured. The grinding/milling process and the Associated machine models must be ordered from the machine manufacturer.

How to Use:

The Hybrid Resin Disc - Zirlux must be handled by the prosthetic professional, who fits the product into a CAD/CAM machining machine, makes the adjustments according to the manufacturer's instructions for use of such machine, therefore, the block will be machined for the purpose of manufacturing of teeth for dental prosthesis.

Precautions:

When processing, use appropriate personal protective equipment (PPE), such as: dust mask, protective gloves, glasses, etc.

Warnings:

The manufacturer recommends single use.

Use only by dental professionals trained in operating CAD/CAM systems;

Avoid overheating during the machining and finishing process;

Dust created during machining can cause irritation to eyes, skin and respiratory tract; It is recommended to use personal protective equipment (mask, glasses);

Ideal temperature for work: Between 20°C and 30°C. To mill the blocks in the CAD/CAM system without an irrigation system, use a maximum of 19,000 RPM to avoid burning the blocks and even damaging the tools.

Storage:

Store away from solvents and intense heat.

Composition:

The product is composed of: Polymer 60~70% (PolyMethylmethacrylate, Dibenzoyl Peroxide, TEGMA, Carboxymethylcellulose), Silanized inorganic glass nanoparticles 30% - 40% (Alumina, Polysiloxane, silicon dioxide), Pigments 0.5% - 1 %.

Validity: 10 years after manufacturing.

Technical Responsible: Rafael Gomes - CRQ: 04263558

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